**FOOD INTOLERANCE  
INTOLLERANZE ALIMENTARI**

**ELITRACE  
ELISA IgG TEST  
ELISA IgG4 TEST**

## Elitrace kit

ELITRACE is an immunoassay kit for quantitative analysis of human serum IgG-IgG4 against food proteins. Food antigens, separately adsorbed on well surface, react with serum IgGs, if present in the sample. After washing to remove unbound molecules, the captured IgGs or IgG4 are recognized by an enzyme-labeled secondary antibody, that then catalyzes its chromogenic substrate. The colorimetric reaction is compared against a human IgG standard curve.

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## Procedure

- Dispense predilute sample
- Incubation 60 minutes at room temperature
- 3 wash cycles
- Dispense anti-human IgG-IgG4 Conjugate
- Incubation 60 minutes at room temperature
- 3 wash cycles
- Dispense chromogenic substrate
- Incubation 10 minutes at room temperature
- Dispense stop solution
- Read at 450 nm (second wavelength 600-650 nm) using linear regression calibration curve

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Quantitative test, calibration curve at 4 points

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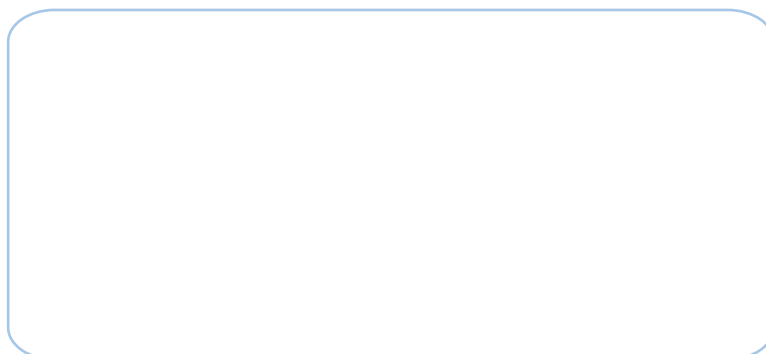
## FOOD LIST

Albumen	Cabbage	Cow's milk	Malt	Peach	Spinach
Almond	Carrot	Cucumber	Mandarin	Peanut	Strawberry
Ananas	Cauliflower	Eggplant	Marjoram	Pear	Sugar cane
Anchovy	Celery	French bean	Melon	Pepper	Sunflower
Apple	Chard	Garlic	Mix berries	Plum	Sweet pepper
Apricot	Cherry	Grapefruit	Mix Cephalopods	Pork	Tea
Artichoke	Chicken	Grapes	Mix Crustaceans	Potato	Tomato
Asparagus	Chickpea	Hake	Mix Mushrooms	Pumpkin	Trout
Baking powder	Cinnamon	Hazelnut	Mustard	Rabbit	Tuna
Banana	Clam	Honey	Oats	Rice	Turkey
Barley	Cocoa	Kiwi	Olive	Rye	Walnut
Bean	Coconut	Kola nut	Onion	Salmon	Watermelon
Beef	Cod	Lamb	Orange	Sheep's milk	Wheat
Brewer's yeast	Coffee	Lemon	Parsely	Sole	Yogurt
Broccoli	Corn	Lentil	Pea	Soy	Yolk

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DISTRIBUTED BY / DISTRIBUITO DA:

**MADE IN ITALY**

MANUFACTURED BY  
**MICROTRACE DIAGNOSTICI**  
 Via IV NOVEMBRE, 84  
 47863 - NOVADELTRIA (RN) - ITALY  
 PHONE +39 0541 92068  
 EMAIL office@microtrace.it